ALCOHOL BREATHALYZER ALCOOTEST

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FEATURES:
LCD Digital Display
Power ON/OFF Button
Breath Portal
Battery Compartment
Durable Carry Strap

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BEFORE USING

1. OPEN battery compartment by removing lid on back of Breathalyzer (Fig. B). Push down on arrow and slide cover off.
2. Insert 2 "AAA" 1.5V high quality Alkaline Batteries (not included) into correct polarity positions shown on Fig. B below. When batteries are low, the L symbol will flash on the LCD screen.

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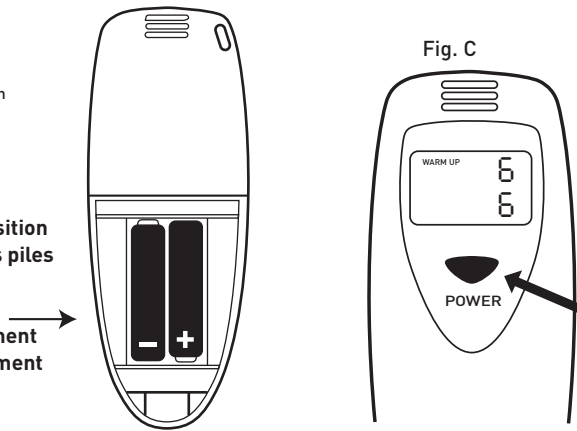
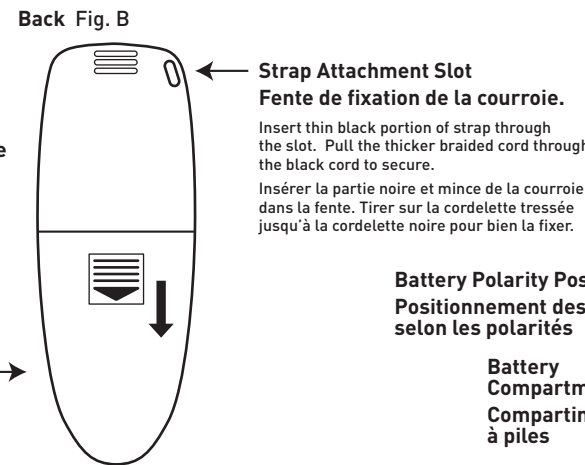
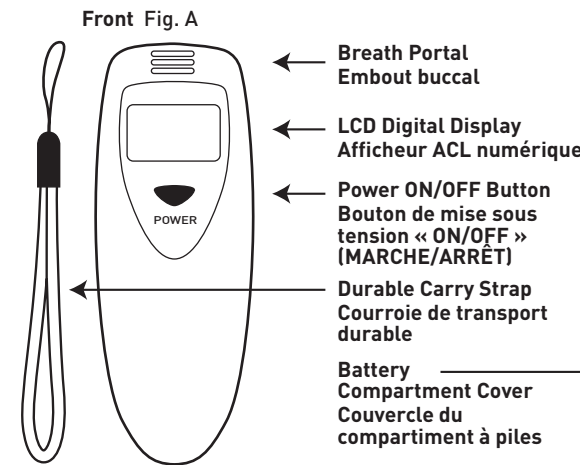
OPERATING INSTRUCTIONS:

IMPORTANT: Wait 15 to 20 minutes after eating, drinking alcohol, or smoking. Alcohol in your mouth could give you an abnormally high reading.


WARM UP MODE:

1. Press and hold the POWER button for 1 second until the breathalyzer beeps.
2. The "WARM UP" symbol will display and a 10 to 00 countdown will start to indicate that the unit is in warm up mode. DO NOT blow into the breathalyzer during warm up.
3. The countdown may repeat one or more times, going from 05 to 00 if the sensor has not warmed up sufficiently. Warm up may take up to 25 seconds. Wait until you see "BLOW" on the left side of the LCD screen.

TESTING MODE:

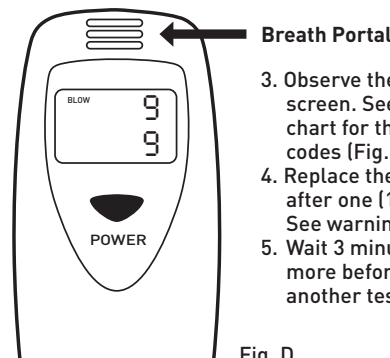1. The "BLOW" symbol will display after the warm up is complete. Another 10 to 00 countdown will start indicating the breathalyzer is in testing mode. If you do not blow into the sensor in the amount of time, an additional 6 seconds will be added to the testing time.
2. Take a deep breath and exhale into the sensor at the top of the unit for 3 to 5 seconds slowly and consistently. The test results will occur when the 10 to 00 countdown is complete. DO NOT touch the top of the unit with your lips.

3. Observe the LCD screen. See the chart for the LCD codes (Fig. E).
4. Replace the unit after one (1) year. See warnings.
5. Wait 3 minutes or more before running another test.

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INTENDED USE

This device is intended to measure the alcohol (%BAC) in the human breath. Measurements obtained by this device should be used as an aid in the detection of alcohol intoxication.

BAC means "blood alcohol concentration". g/l is the percent weight (g)/volume (l).

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
Fig. E

The "CAUTION" symbol is displayed if the alcohol concentration is at 0.05% to 0.08% BAC. The "DANGER" symbol is displayed with an audible beep if the alcohol concentration is 0.08% BAC or higher.

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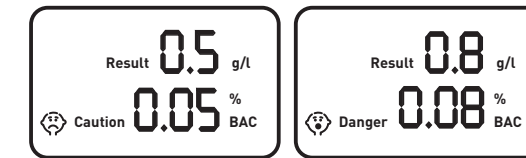
CAUTION: Your driving may be impaired at ANY level of alcohol consumption. DON'T DRINK AND DRIVE!

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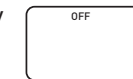AUTO SHUT OFF: The testing results will remain on the display for 15 seconds. After 15 seconds, the "OFF" symbol will appear on the screen for 2-3 seconds and the breathalyzer will turn off.

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WARNING: DO NOT mix old and new batteries. DO NOT mix alkaline, standard (carbon-zinc) or rechargeable (nickel-cadmium) batteries. DO NOT dispose of batteries in fire. Batteries may explode or leak. Remove batteries for long term storage. Batteries could leak causing corrosion.

WARNING: Testing has shown that this device functions accurately for at least 400 tests. Based on NIAAA studies, (www.alcoholism.about.com) this corresponds to one year of device use for most drinkers. YOU SHOULD REPLACE THIS DEVICE ONE (1) YEAR AFTER FIRST USE. FAILURE TO DO SO MAY RESULT IN INACCURATE READINGS. If you are a heavy drinker or heavy smoker, we strongly recommend replacing the unit sooner.

The alcohol breathalyzer does not guarantee 100% accuracy or the same result of the breathalyzers used by law enforcement officials. Test results are guides only and should be not be relied upon. Results can be affected by the environmental conditions, direct sunlight, excessive heat, submersion in liquids, improper usage, elapsed time after drinking, amount of food consumption, medication and other drugs consumed. It is strongly recommended that no motor vehicle or equipment be used after consuming any amount of alcohol, regardless of results of the Alcohol Breathalyzer. The unit uses an LCD screen to deliver test results.

SPECIFICATIONS: Test method: Semi-conductor gas sensor to detect alcohol on breath and converts to BAC level. Preheat time (warm up): 10-25 seconds. Battery life: About 200 tests with "AAA" alkaline batteries. Sevice life: One (1) year or approx. 400 tests. Operating temperature: 10 to 40°C. Storage temperature: 0 to 50°C.

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© 2016 JLR Gear - Made in China/Fabriquée en Chine